

How to use Dissolvo Bags

(Standard instructions for RED Dissolvo Bags)

1. IN STORAGE

- Keep bags sealed within their box until they are needed for use.
- Once a bag is removed from the pack, keep out of sunlight.



2. IN USE

- Handle the bags with dry hands only. Avoid any contact with moisture.
- Do not place overly wet items in the bag at any time. Remember the bag is sensitive to moisture in any form. Wet linen can be wrapped in dry linen.

3. AT CLOSURE

- Expel as much excess air as possible before closing the bag, taking care to avoid expelling air in your face. Reduction of trapped air content makes the closed bag a stronger unit for trans-shipment.
- **DO NOT TIE BAG IN KNOT** ❌
(Bag will NOT work correctly if tied in a knot)
- **USE ATTACHED PINK TIE** ✅



- The closed water soluble laundry bag should be placed within another outer Linen bag for movement within and outside the facility. This reduces the chance of puncture or tearing en-route to the laundry.

4. IN THE LAUNDRY

- The Dissolvo bag should be placed in the washing machine. Pre-sorting is not necessary because the soiled linen is assumed to be contaminated.



- During the initial cold flush cycle, the bag will open allowing contact with the linen by the wash water.
 - Do not overload the washer beyond its rated capacity.
 - Ensure that there is thermal or chemical disinfection during the wash cycle.
 - The water should swirl around the bag linens for 30 seconds before the wash chemicals.
 - Remove red plastic from drum at end of wash before going into dryer (The plastic has been disinfected during the wash cycle).



• Dissolvo Bags Order Code:

- 301 for Red,
- 302 for Yellow
- 304 for Purple.