

# How to use Dissolvo Bags

(Standard instructions for RED Dissolvo Bags)

## 1. IN STORAGE

- Keep bags sealed within their box until they are needed for use.
- Once a bag is removed from the pack, keep out of sunlight.



## 2. IN USE

- Handle the bags with dry hands only. Avoid any contact with moisture.
- **Do not place wet / overly damp linen items in the bag at any time.** Remember the bag is sensitive to moisture in any form. Wet linen can be wrapped in dry linen.

## 3. AT CLOSURE

- Expel as much excess air as possible before closing the bag, taking care to avoid expelling air in your face. Reduction of trapped air content makes the closed bag a stronger unit for trans-shipment.
- **DO NOT TIE BAG IN KNOT** ❌  
( Bag will NOT work correctly if tied in a knot )
- **USE ATTACHED PINK TIE** ✅



- The closed water soluble laundry bag should be placed within another outer Linen bag for movement within and outside the facility. This reduces the chance of puncture or tearing en-route to the laundry.

## 4. IN THE LAUNDRY

- The Dissolvo bag should be placed in the washing machine. Pre-sorting is not necessary because the soiled linen is assumed to be contaminated.



- During the initial cold flush cycle, the bag will open allowing contact with the linen by the wash water.
  - Do not overload the washer beyond its rated capacity.
  - Ensure that there is thermal or chemical disinfection during the wash cycle.
  - The water should swirl around the bag linens for 30 seconds before the wash chemicals.
  - Remove red plastic from drum at end of wash before going into dryer ( The plastic has been disinfected during the wash cycle ).



### • Dissolvo Bags Order Code:

- **301** for Red,
- **302** for Yellow
- **304** for Purple.